---

# REVANESSE® KISS™

---

Instructions for Use  
Gebrauchsanweisung  
Mode d'emploi  
Istruzioni per l'uso  
Instrucciones de uso  
Instruções de Uso  
Инструкция по применению

## Table of Contents:

Symbols .....	4
Assembly of needle to syringe .....	6
Canada .....	8
English .....	8
Français .....	10
EU & Other Countries .....	12
English .....	12
Deutsche .....	14
Français .....	16
Italiano .....	18
Español .....	20
Português .....	22
Русский .....	24
Brazil .....	26
Português .....	26

SYMBOL SYMBOL SYMBOLE SIMBOLO  
 SIMBOLO SIMBOLOGIA СИМВОЛ:

Syringe fluid path sterilized using moist heat  
 Sterilisation des Spritzenflüssigkeitspfades mittels feuchter Hitze  
 Trajectoire du fluide dans la seringue stérilisée à la chaleur humide  
 Percorso del fluido nella siringa sterilizzato a calore umido  
 Canal del fluido de la jeringa esterilizado al vapor  
 Esterilizado por vapor  
 Путь протекающей по шприцу жидкости простерилизован с использованием влажного жара

Store between 2 and 25 °C  
 Bei Temperaturen zwischen 2°C und 25 °C lagern  
 Conserver entre 2 et 25 °C  
 Conservare ad una temperatura compresa tra 2°C e 25°C  
 Almacenar entre 2 y 25°C  
 Armazenar entre 2 e 25 °C  
 Храните при температуре от 2°C до 25°C

Read the Instructions before using the product  
 Lesen Sie die Anweisungen, bevor Sie das Produkt verwenden  
 Lire les instructions avant d'utiliser le produit  
 Leggere le istruzioni prima di utilizzare il prodotto  
 Lea las instrucciones antes de usar el producto  
 Leia as instruções de uso antes de usar o produto  
 Перед использованием изделия прочтите все инструкции

Do not use if the package has been damaged  
 Verwenden Sie das Produkt nicht, wenn die Verpackung beschädigt ist  
 Ne pas utiliser si l'emballage a été endommagé  
 Non utilizzare se la confezione è danneggiata  
 No usar si el paquete se encuentra dañado  
 Não use se a embalagem estiver aberta ou danificada  
 Не используйте, если упаковка повреждена

Do not reuse  
 Das Produkt darf nicht wiederverwendet werden  
 Ne pas réutiliser  
 Evitare il riutilizzo  
 No reutilizar  
 Produto de uso único - proibido reprocessar - não reutilizar  
 Не используйте повторно

Keep away from sunlight  
 Vor Sonneneinstrahlung schützen  
 Tenir à l'écart de la lumière du soleil  
 Non esporre alla luce solare  
 Manter protegido de la luz solar  
 Não expora luz solar  
 Не допускайте попадания солнечного света

Sterilized by irradiation  
 Strahlensterilisation  
 Stérilisé par irradiation  
 Sterilizzato tramite irradiazione  
 Esterilizado por irradiación  
 Esterilizado por irradiação  
 Стерилизован облучением

Keep dry  
 Trocken lagern  
 Garder au sec  
 Conservare in luogo secco  
 Manterner seco  
 Mantenha seco  
 Храните в сухом месте

Lot number  
 Chargennummer  
 Numéro de lot  
 Numero di lotto  
 Número de LOTE  
 Número do lote  
 Номер партии

Expiration date  
 Ablaufdatum  
 Date d'expiration  
 Data di scadenza  
 Fecha de caducidad  
 Data de validade  
 Срок годности

Date of Manufacture  
 Herstellungsdatum  
 Date de fabrication  
 Data di produzione  
 Fecha de fabricación  
 Data de Fabricação  
 Дата изготовления

Manufacturer  
 Hersteller  
 Fabricant  
 Produttore  
 Fabricante  
 Fabricante  
 Производитель

SYRINGE  
 SPRITZE  
 SERINGUE  
 SIRINGA  
 JERINGA  
 SERINGA  
 ШПРИЦ

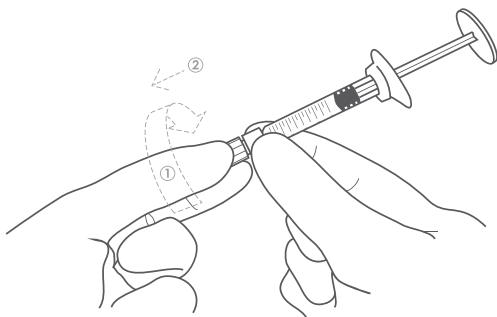
NEEDLE  
 NADEL  
 AIGUILLE  
 AGO  
 AGUJA  
 AGULHA  
 ИГЛЫ

IMPORTER IMPORTEUR IMPORTATEUR IMPORTATORE  
 IMPORTADOR IMPORTADOR ИМПОРТЕР:

Emergo Europe B.V.  
 Prinsessegracht 20  
 2514 AP The Hague  
 The Netherlands

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#### ASSEMBLY OF NEEDLE TO SYRINGE:

1. Carefully unscrew the syringe end cap.
2. Firmly grip the wider part of the needle guard.
3. Press and rotate the needle guard by 90° (one quarter turn). This quarter turn is necessary to lock the needle into the syringe. Remove the needle guard.
4. Using the thumb and forefinger, tightly secure the glass syringe cylinder and the Luer-lock adapter.
5. Grip the needle guard with the opposite hand. To facilitate proper assembly, push and rotate firmly.

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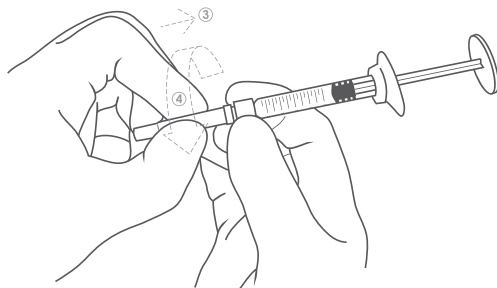
## COMPOSITION

Cross-linked hyaluronic acid .....	25mg/ml
In phosphate buffered saline (Cross-linked with Butanediol-diglycidylether (BDDE))	

## DESCRIPTION

Revasense® Kiss™ is a colorless, odorless, transparent and aqueous gel of synthetic origin. The gel is stored in a pre-filled disposable syringe. Each box contains two 1.0ml syringes of Revasense® Kiss™ along with two sterilized needles.

## APPLICATION RANGE / INDICATIONS

Revasense® Kiss™ is a cross-linked hyaluronic acid gel that is indicated for the treatment of facial rhytides and lipatrophy by injection into the skin. Implantation life is dependent on depth and location, and averages 6-12 months.

## ANTICIPATED SIDE EFFECTS

Physicians must inform patients that with every injection of Revasense® Kiss™ there are potential adverse reactions that may be delayed or occur immediately after the injection. These include but are not limited to:

- Injection-related reactions might occur, such as transient erythema, swelling, pain, itching, discoloration or tenderness at the injection site. These reactions may last for one week.
- Nodules or induration are also possible at the injection site.
- Poor product performance due to improper injection technique.

- Glabellar necrosis, abscess formation, granulomas and hypersensitivity have all been reported with injections of hyaluronic acid products. It is important for physicians to take these reactions into account on a case by case basis.

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## CONTRAINDICATIONS

- Do not inject Revasense® Kiss™ into eye contours (into the eye circle or eyelids).
- Pregnant women or lactating women should not be treated with Revasense® Kiss™.
- Revasense® Kiss™ is only intended for intradermal use and must not be injected into blood vessels. This may occlude and could cause an embolism.
- Patients who develop hypertrophic scarring should not be treated with Revasense® Kiss™.
- Patients that have a known history of streptococcal disease.
- Never use Revasense® Kiss™ in conjunction with a laser, intense pulsed light, chemical peeling or dermabrasion treatments.
- People under the age of 18 should not be treated with Revasense® Kiss™.
- Patients with acne and / or other inflammatory diseases of the skin should not be treated with Revasense® Kiss™.
- Patients with unattainable expectations.
- Patients with multiple severe allergies.
- Patients with acute or chronic skin disease in or near the injection sites.
- Coagulation defects or under anti-coagulation therapy.
- Patients with sensitivity to hyaluronic acid.

It is imperative that patients with adverse inflammatory reactions that persist for more than one week report this immediately to their physician. These conditions should be treated as appropriate (ie: corticosteroids or antibiotics). All other types of adverse reactions should be reported directly to the authorized distributor of the Revasense® family of products and / or to Prollemium Medical Technologies Inc. directly.

## ADMINISTRATION &amp; DOSAGE

- Revasense® Kiss™ should only be injected by or under the direct supervision of qualified physicians who have been trained on the proper injection technique for filling facial wrinkles.
- Before patients are treated they should be informed of the indications of the device as well as its contraindications and potential undesirable side effects.
- The area to be treated must be thoroughly disinfected. Be sure to inject only under sterile conditions.
- Inject the product slowly and apply the least amount of pressure necessary.
- Revasense® Kiss™ and needles packaged with it are for single use only. Do not re-use. If re-used, there is a risk of infection or transmission of blood born diseases.
- Keep the product at room temperature for 30 minutes prior to injection.
- If the skin turns a white color (blanching), the injection should be stopped immediately and the area should be massaged until the skin returns to its normal color.
- Before injecting, press on the plunger of the syringe until a small drop is visible at the tip of the needle.

## PRECAUTIONS

- Revasense® Kiss™ should not be injected into an area that already contains another filler product as there is no available clinical data on possible reactions.
- Revasense® Kiss™ should not be injected into an area where there is a permanent filler or implant.

• Hyaluronic acid products have a known incompatibility with quaternary ammonium salts such as benzalkonium chloride. Please ensure that Revasense® Kiss™ never comes into contact with this substance or medical instrumentation that has come into contact with this substance.

- Revasense® Kiss™ should never be used for breast enlargement, or for implantation into bone, tendon, ligament or muscle.
- Avoid touching the treated area for 12 hours after injection and avoid prolonged exposure to sunlight, UV, as well as extreme cold and heat.
- Until the initial swelling and redness have resolved, do not expose the treated area to intense heat (e.g. solarium and sunbathing) or extreme cold.
- If you have previously suffered from facial cold sores, there is a risk that the needle punctures could contribute to another eruption of cold sores.
- If you are taking aspirin, non-steroidal anti-inflammatory medications, St. John's Wort, High doses of Vitamin E supplements or any similar medications prior to treatment, be aware that these may increase bruising and bleeding at the injection site.
- In order to minimize the risks of potential complications, this product should only be used by health care practitioners who have appropriate training, experience, and who are knowledgeable about the anatomy at and around the site of injection.
- Health care practitioners are encouraged to discuss all potential risks of soft tissue injection with their patients prior to treatment and ensure that patients are aware of signs and symptoms of potential complications.

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- Introduction of product into the vasculature may lead to embolization, occlusion of the vessels, ischemia, or infarction.
- Rare but serious adverse events associated with the intravascular injection of soft tissue fillers in the face have been reported and include temporary or permanent vision impairment, blindness, cerebral ischemia or cerebral hemorrhage, leading to stroke, skin necrosis, and damage to underlying facial structures.
- Immediately stop the injection if a patient exhibits any of the following symptoms, including changes in vision, signs of a stroke, blanching of the skin, or unusual pain during or shortly after the procedure.
- Patients should receive prompt medical attention and possibly evaluation by an appropriate health care practitioner specialist should an intravascular injection occur.

Physicians should take extra precautions when treating patients with autoimmune disorder, HIV, AIDS or who are under immuno-therapy as these patients are at higher risk of adverse events.

**NOTE:** The correct injection technique is crucial to treatment success & patient satisfaction. Revasense® Kiss™ should only be injected by a practitioner qualified according to local laws and standards.

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## SHELF LIFE &amp; STORAGE

Expiry is indicated on each individual package. Store between 2°-25°C, and protect from direct sun light and freezing.

## MANUFACTURER

Prollemium Medical Technologies, Inc.  
138 Industrial Parkway North, Aurora, ON  
L4G 4C3, Canada

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## FABRICANT

Prolenium Medical Technologies, Inc.  
138 Industrial Parkway North, Aurora, ON  
L4G 4C3, Canada

## COMPOSITION

Cross-linked hyaluronic acid ..... 25mg/ml  
 In phosphate buffered saline  
 (Cross-linked with Butanediol-diglycidylether (BDDE))

## DESCRIPTION

Revasse® Kiss™ is a colorless, odorless, transparent and aqueous gel of synthetic origin. The gel is stored in a pre-filled disposable syringe. Each box contains two 1ml syringes of Revasse® Kiss™ along with two sterilized needles.

## APPLICATION RANGE / INDICATIONS

Medical Indications: The products are space-occupying tissue reconstructive materials composed of a hyaluronic acid gel that is indicated for restoration of volume lost from lipatrophy / lipodystrophy, and / or correction of contour deficiencies and anatomic deformities of either pathologic origin or after trauma, in soft tissue.

Intended patients are those desiring the correction of contour deficiencies and deformities in soft tissue, such as HIV-associated lipatrophy and lipodystrophy.

Concomitant Indication: Revasse® Kiss™ is indicated for the treatment of facial rhytides, volume restoration, lip augmentation, skin hydration and contouring of depressions by injection into tissue.

## ANTICIPATED SIDE EFFECTS

Physicians must inform patients that with every injection of Revasse® Kiss™ there are potential adverse reactions that may be delayed or occur immediately after the injection. These include but are not limited to:

- Injection-related reactions might occur, such as transient erythema, swelling, pain, itching, discoloration or tenderness at the injection site. These reactions may last for one week.
- Nodules or induration are also possible at the injection site.
- Poor product performance due to improper injection technique.
- Gelabular necrosis, abscess formation, granulomas and hypersensitivity have all been reported with injections of hyaluronic acid products. It is important for physicians to take these reactions into account on a case by case basis.

Reactions thought to be of hypersensitivity in nature have been reported in less than one in every 1500 treatments. These have consisted of prolonged erythema, swelling and induration at the implant site.

These reactions have started either shortly after injection or after a delay of 2 – 4 weeks and have been described as mild or moderate, with an average duration of 2 weeks. Typically, this reaction is self-limiting and resolves spontaneously with time. However, it is imperative that patients with hypersensitivity type reactions contact their physician immediately for assessment. Patients with multiple allergic reactions should be excluded from the treatment.

## CONTRAINDICATIONS

- Do not inject Revasse® Kiss™ into eye contours (into the eye circle or eyelids).
- Pregnant women or lactating women should not be treated with Revasse® Kiss™.
- Revasse® Kiss™ is only intended for intradermal use and must not be injected into blood vessels. This may occlude and could cause an embolism.
- Patients who develop hypertrophic scarring should not be treated with Revasse® Kiss™.
- Patients that have a known history of streptococcal disease.
- Never use Revasse® Kiss™ in conjunction with a laser, intense pulsed light, chemical peeling or dermabrasion treatments.
- People under the age of 18 should not be treated with Revasse® Kiss™.
- Patients with acne and / or other inflammatory diseases of the skin should not be treated with Revasse® Kiss™.
- Patients with unattainable expectations.
- Patients with multiple severe allergies.
- Patients with acute or chronic skin disease in or near the injection sites.
- Coagulation defects or under anti-coagulation therapy.
- Patients with sensitivity to hyaluronic acid.

It is imperative that patients with adverse inflammatory reactions that persist for more than one week report this immediately to their physician. These conditions should be treated as appropriate (ie: corticosteroids or antibiotics). All other types of adverse reactions should be reported directly to the authorized distributor of the Revasse® family of products and / or to Prollenium Medical Technologies Inc. directly.

## ADMINISTRATION &amp; DOSAGE

- Revasse® Kiss™ should only be injected by or under the direct supervision of qualified physicians who have been trained on the proper injection technique for filling facial wrinkles.
- Before patients are treated they should be informed of the indications of the device as well as its contraindications and potential undesirable side effects.
- The area to be treated must be thoroughly disinfected. Be sure to inject only under sterile conditions.
- Inject the product slowly and apply the least amount of pressure necessary.
- Revasse® Kiss™ and needles packaged with it are for single use only. Do not re-use. If re-used, there is a risk of infection or transmission of blood born diseases.
- Keep the product at room temperature for 30 minutes prior to injection.
- If the skin turns a white color (blanching), the injection should be stopped immediately and the area should be massaged until the skin returns to its normal color.
- Before injecting, press on the plunger of the syringe until a small drop is visible at the tip of the needle.

## PRECAUTIONS

- Revasse® Kiss™ should not be injected into an area that already contains another filler product as there is no available clinical data on possible reactions.
- Revasse® Kiss™ should not be injected into an area where there is a permanent filler or implant.
- Hyaluronic acid products have a known incompatibility with quaternary ammonium salts such as benzalkonium chloride. Please ensure that Revasse® Kiss™ never comes into contact with this substance or medical instrumentation that has come into contact with this substance.
- Revasse® Kiss™ should never be used for breast enlargement, or for implantation into bone, tendon, ligament or muscle.
- Avoid touching the treated area for 12 hours after injection and avoid prolonged exposure to sunlight, UV, as well as extreme cold and heat.
- Until the initial swelling and redness have resolved, do not expose the treated area to intense heat (e.g. solarium and sunbathing) or extreme cold.
- If you have previously suffered from facial cold sores, there is a risk that the needle punctures could contribute to another eruption of cold sores.
- If you are taking aspirin, non-steroidal anti-inflammatory medications, St. John's Wort, High doses of Vitamin E supplements or any similar medications prior to treatment, be aware that these may increase bruising and bleeding at the injection site.
- In order to minimize the risks of potential complications, this product should only be used by health care practitioners who have appropriate training, experience, and who are knowledgeable about the anatomy and around the site of injection.
- Health care practitioners are encouraged to discuss all potential risks of soft tissue injection with their patients prior to treatment and ensure that patients are aware of signs and symptoms of potential complications.

## WARNINGS

Confirm that the seal on the box has not been broken and sterility has not been compromised. Confirm that the product has not expired. Product is for single use only; do not re-use. If re-used, there is a risk of infection or transmission of blood born diseases.

Introduction of product into the vasculature may lead to embolization, occlusion of the vessels, ischemia, or infarction.

- Rare but serious adverse events associated with the intravascular injection of soft tissue fillers in the face have been reported and include temporary or permanent vision impairment, blindness, cerebral ischemia or cerebral hemorrhage, leading to stroke, skin necrosis, and damage to underlying facial structures.
- Immediately stop the injection if a patient exhibits any of the following symptoms, including changes in vision, signs of a stroke, blanching of the skin, or ureal pain during or shortly after the procedure.
- Patients should receive prompt medical attention and possibly evaluation by an appropriate health care practitioner specialist should an intravascular injection occur.

Physicians should take extra precautions when treating patients with auto immune disorder, HIV, AIDS or who are under immuno-therapy as these patients are at higher risk of adverse events.

**NOTE:** The correct injection technique is crucial to treatment success & patient satisfaction. Revasse® Kiss™ should only be injected by a practitioner qualified according to local laws and standards.

The graduation on the syringe is not precise and should be used as a guide only. The amount of material to be injected is best determined by visual and tactile assessment by the user.

## SHELF LIFE &amp; STORAGE

Expiry is indicated on each individual package. Store between 2°-25° C, and protect from direct sun light and freezing.

## MANUFACTURER

Prollenium Medical Technologies, Inc.  
 138 Industrial Parkway North, Aurora, ON  
 L4G 4C3, Canada

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ProLium Medical Technologies, Inc.